



Infusions Made Easy

The SCIg60™ Infuser is a reusable constant pressure mechanical syringe pump controlled by a proprietary variable rate flow device.

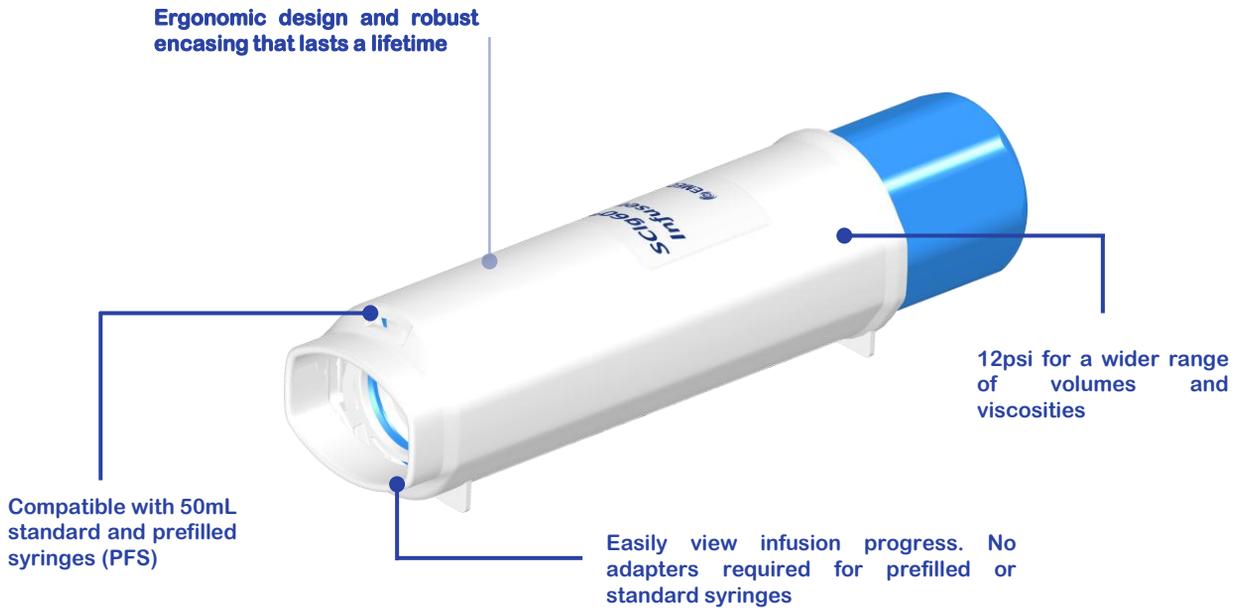

Highlights

SCIg60™ can be paired with the VersaRate® flow controller, a disposable device that offers the clinician and patient a way to adjust the flow rate with an easy-to-use dial.

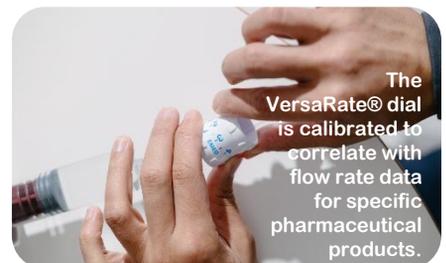

Quick Reference

See instructions for use for complete guide.



UNTWIST to open the pump and prepare for syringe loading.



LOAD either a standard or prefilled 50mL syringe into the infuser and twist clockwise to secure in place.



TWIST to load the infuser and prepare for infusion. The infuser is validated for over 4,200 uses.



BEGIN after priming and infusion site preparation per the steps in the IFU. Begin your infusion and leverage the carrying case as needed to enhance mobility.

Features

Portable, lightweight and quiet

Mechanically powered

Designed for use with the proprietary VersaRate® flow controller or Infuset™ fixed rate sets

Benefits

SClg60™ connected with the VersaRate® flow controller puts variable rate flow in the hands of the clinician and patient providing optimal infusion therapy

Easy for clinician to train patient for home use

Infusets™ are available for special situations that require a single rate

Advantages

Offers control of infusion therapy without the cost of an electronic pump

SClg60™, when used in conjunction with the VersaRate®, gives patients flexibility and can replace rate control sets

Flow rate charts are available for specific applications

Ordering Information

For orders and inquiries, contact your local EMED representative or EMED's customer service department at 916-932-0071. Orders can also be placed via email at orders@emedtc.com.

EMED Reorder Number

Item Description

Qty Per Box

FP-0010002

SClg60 Mechanical Infusion Pump (50mL)

1